

The Uterine Sound

The Uterine Sound is used to gauge the depth and position of the uterine cavity. It is also helpful with the placement of IUDs and the verification of the intrauterine position and length for HSG procedures. The proximal portion of the sound is 3.5 mm in diameter tapering to 2.5 mm at the distal portion with a bulbous tip to reduce the risk of perforation. Malleable to adapt to anatomical variations. It is a sterile, disposable, single-use only device.



Product Number

MID-100US

To learn more about all that Murph Industries has to offer, please visit our website, www.murphind.com or you can speak to a direct sales representative right away by calling 610-671-3900 or emailing us at sales@murphind.com

DIRECTION FOR USE: The Uterine Sound is a single-use sound. It is made of white, malleable, plastic material. Prior to use, the sound can be curved between the thumb and finger to follow the curve of the cervical canal. The graduations are in centimeters and allow for the measurement of the uterine cavity. Dispose of the Uterine Sound in accordance with all applicable Federal, State, and local Medical / Hazardous waste practice

PRECAUTIONS: Do not use if package seal is damaged or broken. Minor bleeding, cervical inflammation, and abdominal cramps may occur.

CONTRAINDICATIONS: This disposable Uterine Sound should not be used in the following cases: ● Possible pregnancy ● Infection, including but not limited to Acute Cervicitis and Endometriosis ● Unexplained Uterine Bleeding

WARNINGS: Gently introduce and manipulate the tip of the sound through the cervical canal. Forced entry could create a false tunnel and bleeding.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician. Do not reuse for avoiding user may be infected.

















Wellkang Ltd Suite B, 29 Harley Street LONDON, W1G 9QR, U.K.





Not made with natural rubber latex (LATE)



NOT MADE WITH NATURAL RUBBER LATEX